


Blood Glucose and Lactate Meter

Lactate Measurement, diagnostic
and training planning



Lactate[®]
EXPRESS

Reasons to Buy

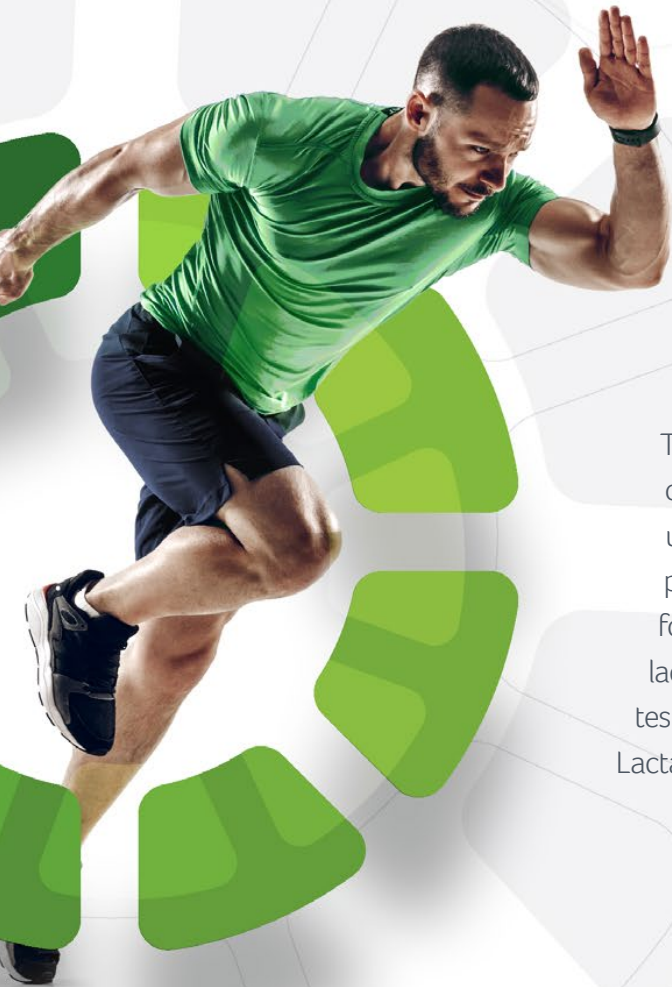

Lactate Express comes with **mesics**[®] lactate kinetics App for your entry into professional lactate performance diagnostic.

Take your training to the next level, whether as an individual athlete or with your entire team. The Lactate Express App offers unwavering support with its array of professional reports and personalized training plans. So remarkable, they're not just valuable for your progress, but they're also marketable. Create complete lactate diagnostics in under 5 minutes. Enter your determined stage test data into the App, select a lactate model and a training plan. Lactate Express takes care of the rest. Your report is ready.

- Blood Glucose and Lactate Meter



4 2 7 0 0 0 3 9 5 4 9 1 4

With a compact design and unparalleled price-performance ratio, it excels in various settings, including outdoor training, gym workouts, and laboratory analysis.

- Test Strips (50pcs)



4 2 7 0 0 0 3 9 5 4 9 2 1

PRECISE AND FAST RESULT IN

10s

Requiring only a minimal 0.8 μ l blood sample.

Blood Glucose and Lactate Meter



How to

1



Take out a test strip and avoid touching the sensor surface.

2



Insert the test strip. Puncture your finger or earlobe and collect blood with the sensor.

3



Result appears within the blink of an eye.

4



Finally, remove. Save without any need to touch the sensor (contactless)


5

Data-Sync into the App



6

Evaluation / Report (generated by App)

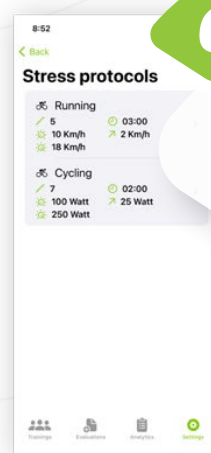

Instrument Parameters and Distributor Information

Subject	Specification
Technology type	Electrochemical biosensor technology
Range of measurement	Blood glucose: 0.6 ~ 33.3 mmol/L; Blood lactate: 0.5 ~ 28.0 mmol/L
Sample types	Fresh fingertip capillary and venous whole blood
Blood sample requirement	0.8 μ L
Measuring time	Blood glucose result in 5 sec, Blood lactate result in 10 sec
Hematocrit range	Blood glucose test: HCT range 10% to 65% Blood lactate test: HCT range 30% to 60%
Memory capacity	800 sets
Operating conditions	Blood glucose test: 10°C ~ 40°C, Blood lactate test: 15°C ~ 35°C RH: 10% ~ 85%, 700 hPa ~ 1066 hPa
Device service life	2 years
Dimensions	240 mm x 120 mm x 110 mm
Auto strip ejection	Prevents any contact with blood sample

mesics GmbH

Schreiberstraße 24
48149 Muenster
Germany
Tel: +49 (0) 251 2026 2005

Download on the
App Store



www.lactate-express.com